Manufactured by:


EYELINE OPTICAL

PO Box 14 341, Kilbirnie, Wellington, New Zealand. Tel: +64 4 386 3637 Fax: +64 4 386 1214 Toll free: 0508 11 44 77 Fax free: 0800 11 44 77 E-mail: enquiries@eyeline.co.nz

Distributed by

BOTT CONFRONTATIONAL FIELD TARGET

Also known as the BOTT CONFRONTATIONAL WAND

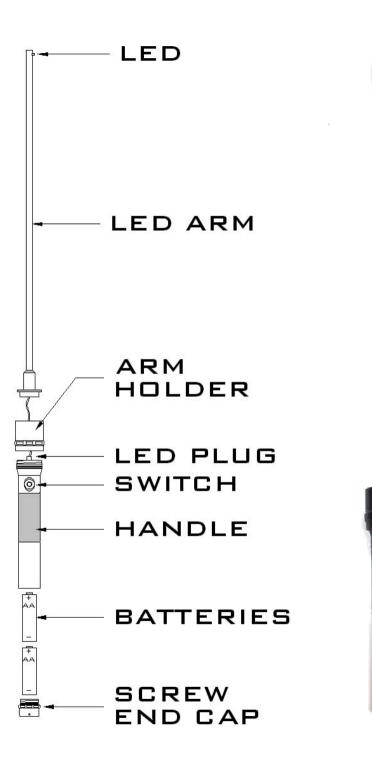

BOTT CONFRONTATION FIELD TARGET - Mk II

(also known as the) BOTT CONFRONTATION WAND

The Bott confrontation field target was designed by New Zealand Optometrist Alan Bott who recognised the need for a small reliable low cost target to assess the visual fields of patients.

It has been well documented that a flashing light of know intensity and size is a suitable target for visual field testing. It is the purpose of this device to provide a suitable target when assessing fields by confrontation.

The quality aluminium handle fits comfortably into the hand. Two AA size batteries power the instrument. The handle supports a 330mm long black wand which contains the flashing light. The light is fitted into the side of the wand which allows it to be easily turned away from the subject to allow for false positive tests.

Directions

- Place subject directly in front of examiner in subdued light,
- Patient covers one eye with card and directed to watch the examiners opposite eye (ie: if testing the right eye the subject is asked to look at the examiners left eye),
- The device is held in the blind area beyond the subject's field of view and is switched on,
- The wand is slowly brought forward and the subject is asked to say when they see the flashing light while still looking at the examiner,
- The extent of the field can be estimated by the examiner,
- The device can be used as a target for the Bjerrum Screen for assessment of gross areas of field loss.

Should the examiner feel that the subject either does not understand or is commenting when he sees something other than the light, then the wand can be rotated through 180° which will prevent the light from being seen by the subject. If the examiner still gets a response, he will know he has a false positive and will need to re-instruct and test the subject.