

ADD-A-PADS



ADD-A-PAD KIT CONTAINS


1 x bottle of acetone adhesive (0.5 oz)
3 pairs each, thin, medium, heavy and medium straight acetate pads

- Ideal for narrowing plastic frame bridge
- Been used by dispensers for over 40 years - trusted and proven
- Refill items available separately
- Because pads are clear they take on the colour of the frame, thereby not becoming noticeable.



Final appearance

Product Code	Description	Price
PO153	Ad-A-Pad kit set - 16pr assorted size pads & adhesive	\$25.00 set
PO154MIX	Ad-A-Pads, 12prs, mixed sizes, straight, thin, medium & heavy	\$22.50 bx
PO152	Ad-A-Pad Pro Adhesive - 0.5oz Bottle	\$4.95 btl
PO154HEAV	Ad-A-Pad clear acetate bridge pad - Size: Heavy	\$2.50 pr
PO154MED	Ad-A-Pad clear acetate bridge pad - Size: Medium	\$2.50 pr
PO154STR	Ad-A-Pad clear acetate bridge pad - Size: Straight	\$2.50 pr
PO154THIN	Ad-A-Pad clear acetate bridge pad, Size: Thin	\$2.50 pr

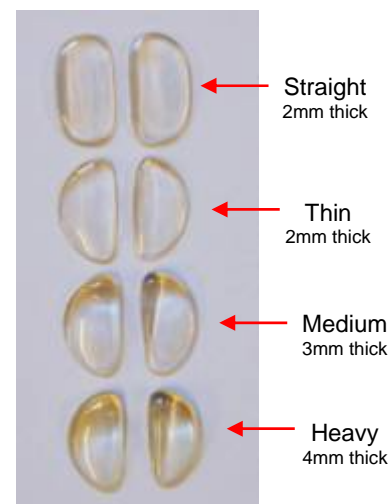

Prices are confidential to the recipient. Prices exclude GST and freight and are subject to change without notice
If you no longer wish to receive our emails click [UNSUBSCRIBE](#), type "unsubscribe: in the subject line, then click SEND

AD-A-PADS - Use & Instructions

- Ideal for narrowing plastic frame bridge
- Been used by dispensers for over 40 years - trusted and proven
- Refill items available separately
- Because pads are clear they take on the colour of the frame, thereby not becoming noticeable.

AD-A-PAD KIT CONTAINS

1 x bottle of acetone adhesive (1/2 oz)

3 pairs each, thin, medium, heavy and medium straight acetate pads

AD-A-PAD KIT




ADHESIVE



ACETATE PADS

(available separately in pairs)
Not to scale

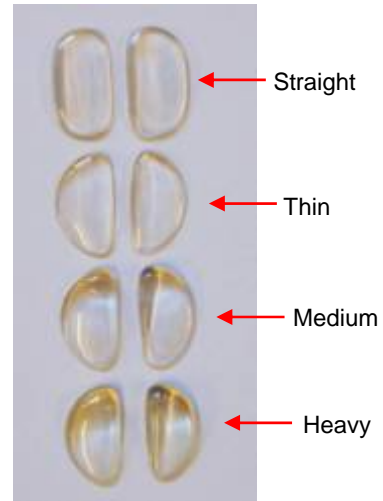

Pad profile
not to scale



INSTRUCTIONS



#1 The Frame



#2 Place the pad on finger and drop in the adhesive until cavity completely filled.



#3 Check position of pad placement - (Practice before adhesive is placed into cavity)



#4 Turn frame vertically and place pad against frame and hold for 30 seconds.

Wipe any surplus adhesive with tissue.

Repeat for second pad,



#5 Final appearance
Leave two hours to harden.